


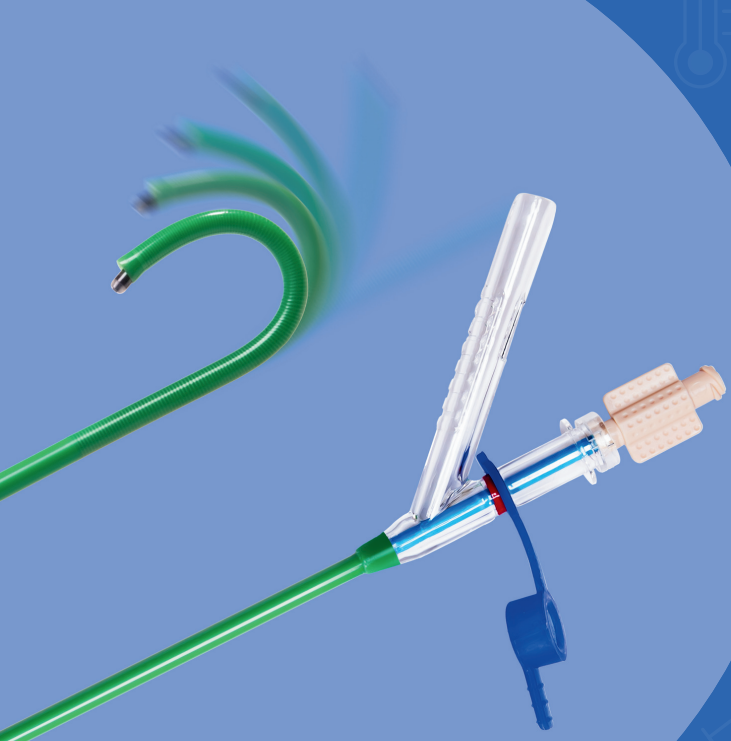

ClearPetra

Next Generation Ureteral Access Sheath

PATENTED Flexible tip ClearPetra allows you to aspirate stones from the pyelocalyceal system.

- **The ClearPetra Ureteral Access Sheath** has an oblique side channel with pressure vent that can be connected to negative pressure aspiration.
- **ClearPetra** now also offers a **FLEXIBLE** tip that can be passively deflected 180 degrees to the lower pole calyces by the flexible ureteroscope thus allowing for efficient and effective treatment of the urinary stone.

How does ClearPetra work?

- The inflow of irrigation is through the endoscope.
- The outflow of irrigation is between the scope and the inner sheath.
- A vortex is created at the distal tip of the sheath.
- The irrigation fluid, stone fragments, dust and blood are caught in the vortex and can be aspirated out down the inner channel.
- The aspiration pressure can be controlled by the pressure vent on the oblique side port of the sheath.

BioSpectrum Ltd

Unit 9 Halifax Court,
Fernwood Business Park,
Newark, Nottinghamshire,
NG24 3JP

Contact us for more information

<http://www.bio-spectrum.co.uk>
 01636 705 084
customers@bio-spectrum.co.uk
https://twitter.com/Bio_Spectrum
<https://www.linkedin.com/company/bio-spectrumuk>
<https://www.youtube.com/channel/UCBtwuYGwGDZld8EA-hWLSXA>

Follow us



ClearPetra

Next Generation Ureteral Access Sheath



Set Up :

- Use a disposable flexible ureteroscope (BioSpectrum offer the Urofino 7.5Fr scope) with the ClearPetra Flexible Ureteral Access Sheath.
- Select a ClearPetra sheath that is at least 2Fr larger than the flexible scope: 7.5Fr scope = 10Fr ClearPetra sheath.
- Set continuous negative aspiration (suction) to 200mmHg.
- Use the continuous pressurised irrigation to 60cc/minute.
- Withdraw the scope to the marked red band on the sheath to open an unimpeded channel for the passage of large stone fragments.

Instructions For Use :

- Position the tip of the sheath as close to the stone as possible.
- Remove inner obturator and place blue rubber cap onto the proximal straight end.
- Connect the sheath to a negative pressure aspirator/suction.
- Insert endoscope.
- Commence lithotripsy using Holmium-Yag laser
- Pressure can be adjusted using the pressure vent on the oblique side port.
- After the surgery is complete, turn off the negative pressure aspirator and send the collection bottle for analysis.

BioSpectrum Ltd

Unit 9 Halifax Court,
Fernwood Business Park,
Newark, Nottinghamshire,
NG24 3JP

Contact us for more information


<http://www.bio-spectrum.co.uk/>
 01636 705 084
customers@bio-spectrum.co.uk
https://twitter.com/Bio_Spectrum
<https://www.linkedin.com/company/bio-spectrumuk>
<https://www.youtube.com/channel/UCBtwuYGwGDZld8EA-hWLSXA>

Follow us



ClearPetra

Next Generation Ureteral Access Sheath

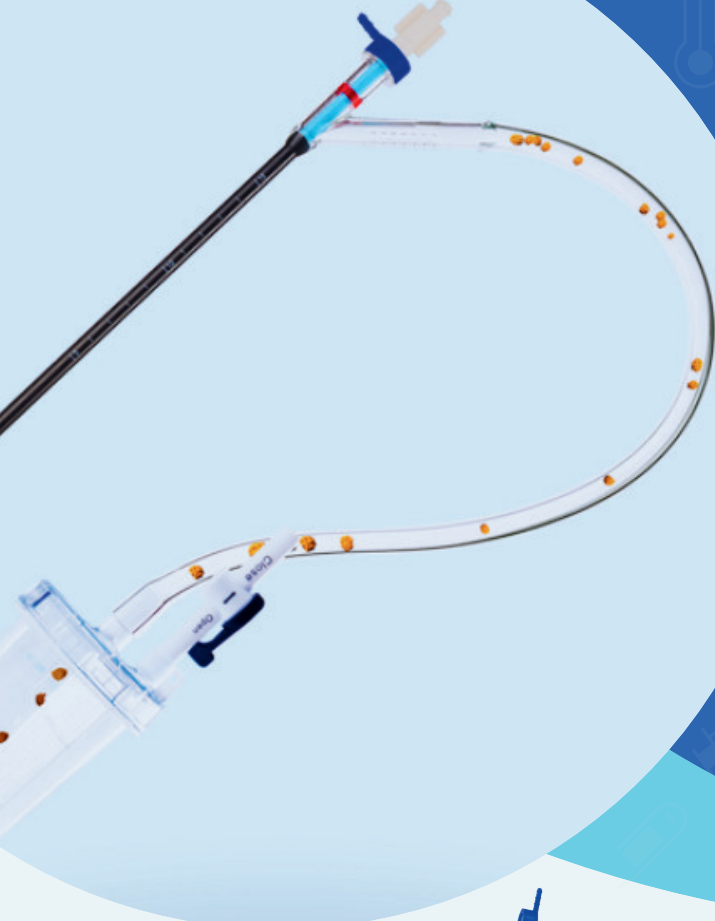

Key Benefits :

- **Maintaining Low Intra-renal Pressure and Temperature** – a vortex is created by the continuous irrigation/suction. The irrigated fluid can be effectively evacuated, thus always maintaining low intra-renal pressure and temperature, ensuring fewer complications and thermal damage.
- **High Stone Clearance** – the flexible tip of the ClearPetra can be placed in the pyelocalyceal system for better aspiration of the stone fragments even in the lower pole.
- **High Surgical Efficacy** – stone dust can be aspirated out during laser lithotripsy thus allowing improved visual field for the surgeon.
- **Reduced Retropulsion** – stone fragments will aggregate at the distal end of the ClearPetra sheath instead of scattering.
- **Treating Larger Stones** – due to the improved visual field surgeons can treat larger stones.
- **Fewer Accessory Devices Needed** – stone baskets, forceps and anti-retropulsion devices are no longer necessary as the stone fragments are aspirated out spontaneously.

BioSpectrum Ltd

Unit 9 Halifax Court,
Fernwood Business Park,
Newark, Nottinghamshire,
NG24 3JP

Contact us for more information

<http://www.bio-spectrum.co.uk/>
 01636 705 084
customers@bio-spectrum.co.uk
https://twitter.com/Bio_Spectrum
<https://www.linkedin.com/company/bio-spectrumuk>
<https://www.youtube.com/channel/UCBtwuYGwGDZld8EA-hWLSXA>

Follow us



Ordering information :

A – ClearPetra FLEXIBLE Ureteral Access Sheath

NHS SC Code	BioSpectrum Code	Description	Size (ID/OD x L)
Size: 9.5Fr			
TBC	90119536	ClearPetra Ureteral Sheath 9.5Fr x 36cm	9.5Fr/11.5Fr x 36cm
TBC	90119540	ClearPetra Ureteral Sheath 9.5Fr x 40cm	9.5Fr/11.5Fr x 40cm
TBC	90119546	ClearPetra Ureteral Sheath 9.5Fr x 46cm	9.5Fr/11.5Fr x 46cm
TBC	90119555	ClearPetra Ureteral Sheath 9.5Fr x 55cm	9.5Fr/11.5Fr x 55cm
Size: 10/12Fr			
FUA030	90111036	ClearPetra FLEXIBLE Sheath 10Fr x 36cm	10Fr/12Fr x 36cm
FUA031	90111040	ClearPetra FLEXIBLE Sheath 10Fr x 40cm	10Fr/12Fr x 40cm
FUA124	90111046	ClearPetra FLEXIBLE Sheath 10Fr x 46cm	10Fr/12Fr x 46cm
FUA044	90111055	ClearPetra FLEXIBLE Sheath 10Fr x 55cm	10Fr/12Fr x 55cm
Size: 11/13Fr			
FUA112	90111136	ClearPetra FLEXIBLE Sheath 11Fr x 36cm	11Fr/13Fr x 36cm
FUA130	90111140	ClearPetra FLEXIBLE Sheath 11Fr x 40cm	11Fr/13Fr x 40cm
FUA115	90111146	ClearPetra FLEXIBLE Sheath 11Fr x 46cm	11Fr/13Fr x 46cm
FUA077	90111155	ClearPetra FLEXIBLE Sheath 11Fr x 55cm	11Fr/13Fr x 55cm
Size: 12/14Fr			
FUA101	90111236	ClearPetra FLEXIBLE Sheath 12Fr x 36cm	12Fr/14Fr x 36cm
FUA102	90111240	ClearPetra FLEXIBLE Sheath 12Fr x 40cm	12Fr/14Fr x 40cm
FUA076	90111246	ClearPetra FLEXIBLE Sheath 12Fr x 46cm	12Fr/14Fr x 46cm
FUA050	90111255	ClearPetra FLEXIBLE Sheath 12Fr x 55cm	12Fr/14Fr x 55cm

B – ClearPetra Nephrostomy Sheath (for Mini PCNL)

NHS SC Code	BioSpectrum Code	Description	Size (ID/OD x L)
FUA049	90121613	ClearPetra Nephrostomy Sheath 16Fr x 13cm	16Fr/18Fr x 13cm
FUA103	90121615	ClearPetra Nephrostomy Sheath 16Fr x 15cm	16Fr/18Fr x 15cm
FUA064	90121617	ClearPetra Nephrostomy Sheath 16Fr x 17cm	16Fr/18Fr x 17cm
FUA079	90121621	ClearPetra Nephrostomy Sheath 16Fr x 21cm	16Fr/18Fr x 21cm
FUA078	90121813	ClearPetra Nephrostomy Sheath 18Fr x 13cm	18Fr/20Fr x 13cm
TBC	90121815	ClearPetra Nephrostomy Sheath 18Fr x 15cm	18Fr/20Fr x 15cm
FUA063	90121817	ClearPetra Nephrostomy Sheath 18Fr x 17cm	18Fr/20Fr x 17cm

C – ClearPetra Cystoscopy Sheath

NHS SC Code	BioSpectrum Code	Description	Size (ID/OD x L)
FUA027	90121821	ClearPetra Nephrostomy Sheath 18Fr x 21cm	18Fr/20Fr x 21cm
FUA123	90131824	ClearPetra Nephrostomy Sheath 18Fr x 24cm	18Fr/20Fr x 24cm
FUA056	90132021	ClearPetra Nephrostomy Sheath 20Fr x 21cm	20Fr/22Fr x 21cm
FUA058	90132024	ClearPetra Nephrostomy Sheath 20Fr x 24cm	20Fr/22Fr x 24cm
FUA041	90132221	ClearPetra Nephrostomy Sheath 22Fr x 21cm	22Fr/24Fr x 21cm
FUA121	90132224	ClearPetra Nephrostomy Sheath 22Fr x 21cm	22Fr/24Fr x 21cm

D – ClearPetra Nephrostomy Sheath With Dilators Set

NHS SC Code	BioSpectrum Code	Description	Size (ID/OD x L)
FUQ85075	U04A010101	ClearPetra 16Fr x 15cm Dilator Set for Mini PCNL (Dilators: 10Fr, 12Fr, 14Fr, 16Fr x 22cm)	16Fr x 15cm
FUQ85074	U04A013132	ClearPetra 18Fr x 15cm Dilator Set for Mini PCNL (Dilators: 10Fr, 12Fr, 14Fr, 16Fr, 18Fr x 22cm)	18Fr x 15cm

BioSpectrum Ltd

Unit 9 Halifax Court,
Fernwood Business Park,
Newark, Nottinghamshire,
NG24 3JP

Contact us for more information

<http://www.bio-spectrum.co.uk/>
 01636 705 084
customers@bio-spectrum.co.uk
https://twitter.com/Bio_Spectrum
<https://www.linkedin.com/company/bio-spectrumuk>
<https://www.youtube.com/channel/UCBtwuYGwGDZld8EA-hWLSXA>

Follow us

